

# 3M™ True Definition Scanner

Doctors with a 3M™ True Definition scanner can send digital impressions directly to Protech Dental Studio through 3M Connection Center.

## To send a file:

- 1) Log in to your 3M Connection Center Account that was created when your scanner was installed. If assistance is needed, contact 3M Digital Impression Customer Service at 877.722.6528.
- 2) Verify that Protech Dental Studio (No Model), are available under the Scanner Laboratory menu.
  - If not, contact 3M Digital Impression Customer Service (877.722.6528) and request adding the profiles to the scanner menu.
- 3) Select the Protech Dental Studio profile from the drop down menu that matches your case preference:
  - Protech Dental Studio (choose this option when requesting models)
  - Protech Dental Studio (choose this option when requesting model-less restorations)
- 4) Complete the intraoral scan and select the Protech Dental Studio profile that meets your model preference.
- 5) Complete the True Definition online Rx.
- 6) Upload the scan data to the 3M Communication Center.