

Carestream CS Scanner

Doctors with a Carestream CS Scanner can send digital impressions directly to **Protech Dental Studio** through CS Connect or Direct E-mail.

To send a file:

- 1)** Complete the intraoral scan and open the patient's case in the DIS software. This software can operate in standalone mode or through Practiceworks.
- 2)** Through the file button, in the upper right corner of the patient history window, select *history open* by clicking the *book icon*.
- 3)** After highlighting the 3DIO folder, click the *Send to* button located toward the lower right corner of the window. The STL files will now be saved to the desktop.

To submit to Protech Dental Studio using CSCONNECT:

- a.** After highlighting the appropriate 3DIO File from step 3, click the *CS connect* button. The web portal will open to login screen.
- b.** Log into CS Connect.
- c.** From the drop down menu, select *Protech Dental Studio*.
- d.** Attach files by clicking *Browse* and selecting the *.dcm file* from the patient's folder; then click *Open*.
- e.** Add any pertinent information regarding the case to the comment box.
- f.** Accept the terms and conditions by checking the box and click *Submit*.