

## To send a file:

- 1)** Create an account on [www.sirona-connect.com](http://www.sirona-connect.com) and follow the prompts for activating the account (if you already have an account, proceed to step 3).
- 2)** After account activation, log into your account and add **Protech Dental Studio** to the list of Favorite Laboratories.
- 3)** Log into your Cerec software and complete the digital scan.
- 4)** After completing scan acquisition and marking the margin, proceed to the Connect step and enter Login info.
- 5)** Review your order and restoration information then click *Enter Order Data*.
- 6)** Once the upload is complete, select *Protech Dental Studio* and desired return date.
- 7)** Click *Add Additional Information* and enter gender, age, comments, and upload any additional files for the case. If model-less, please specify.
- 8)** Review information and click *Submit Cart*.
- 9)** Enter your user name and password once again and the case submission is complete.