

Closed Tray (Indirect) Procedure



1: Screw Indirect Transfers onto implants with the 1.25mmD Hex Tool. The Fixture Mount/Transfer can also be used.



2: For a closed tray (indirect) impression, retaining screws are used that match the height of the transfer.




3: Block out holes on top of the screws with wax or other suitable material to prevent impression material from entering.



4: Record the impression. Place light to medium body impression material around transfer and record a full-arch impression with medium body material.



5: Unscrew each transfer from the implant bodies using the 1.25mmD Hex Tool and attach the appropriate implant replica with the same retaining screw.

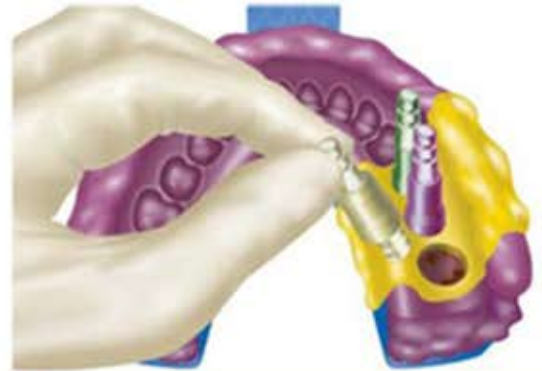

6: Place the transfers back into the impression material with the replicas attached. The impression is now ready to send to the laboratory for fabrication of a stone model.