

To send a file:

iTero® Element™

- 1) Add **Protech Dental Studio** by calling iTero Support 800-577-8767.
- 2) Start a new Rx, and after entering patient name and case type, select *Protech Dental Studio* from the Ship To menu.
- 3) If model-less, specify in the notes.
- 4) After scanning, press the *Envelope icon*, add signature, and press *Confirm & Send*.

iTero® HD2.9 and Previous Models

- 1) Add **Protech Dental Studio** by calling iTero Support at 800-577-8767 or from within your iTero online account.
- 2) Start a new Rx, enter required information and select *Protech Dental Studio* from the Ship To menu.
- 3) If model-less, specify in the notes.
- 4) After completing the scan process, press the *Send* button.