

To send a file:

Dentists should be registered with a 3Shape Communicate account when the scanner is installed. If Protech Dental Studio is not already pre-installed on the list of available labs, send a Connection Request from the Communicate account to Protech Dental Studio.

- 1) After selecting the patient name, press *New Session*.
- 2) The lab list will appear, select *Protech Dental Studio*.
- 3) Complete the order form information. In the Comments, specify specific materials, and if the case is model-less.
- 4) Complete the scanning process and press the *Send Order* envelope icon.
- 5) On the Send Order screen, verify the case information and scans, and then press *Send Order*.